Scan for more information

Advancis
Medical

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Activon® Tulle

Knitted viscosе mesh impregnated with 100% Manuka honey

Instructions for use

Advancis
Medical

Activon® Tulle

Knitted viscose mesh impregnated with 100% Manuka honey

Product description

Activon® Tulle is a knitted viscose mesh primary dressing impregnated with 100% Medical Grade Manuka honey. Activon® Tulle creates a moist healing environment and effectively eliminates wound odour whilst providing antibacterial action. Activon® Tulle is ideally suited to debriding and de-sloughing shallow wounds or where the exudate levels have started to decrease.

Indications

May be used on partial or full thickness wounds including: sloughy wounds, pressure ulcers, leg ulcers, surgical wounds, burns, infected wounds, graft & donor sites, and malodorous wounds.

Directions for use

Apply directly onto wound bed (either side down), can be cut to size. Cover with a secondary dressing. May be unfolded to cover larger areas although this reduces the concentration of honey, efficacy is related to volume of available honey.

Frequency of change

Depending on wound exudate levels, any surrounding interstitial fluid, oedema and dressing regimen may require changing daily but can be left in place for up to 7 days. Will be less effective when diluted by wound exudate. To maintain an effective honey level, change the dressing when the colour changes significantly.

Contra-indications

Contains glucose, monitor the levels of patients with diabetes. Do not use if allergic to bee products. A few patients experience pain when applied, depending on sensitivity of the wound it may be necessary to consider an appropriate level of analgesia. If pain continues unabated, discontinue use and irrigate the wound with saline solution.

Disposal

Dispose as clinical waste.

Storage

Store at room temperature.

Warnings and Precautions

Activon® Tulle is a single use product. If used on more than one patient, cross-contamination or infection may result. Opening the dressing pack compromises the sterile barrier. Any unused dressings should be discarded.

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