Advazorb® Sacral

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Utylizacja

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Advazorb® Sacral

Hydrophilic foam sacral dressing with film backing

Instructions for use

Advazorb® Sacral

Hydrophilic foam sacral dressing with film backing

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Scan for more information

Product Description

A multi-layer sacral wound dressing composed of soft silicone that adheres only to normal healthy skin. A soft, conforming low adherent hydrophilic polyurethane foam wound dressing with breathable film backing. Advazorb® Sacral has high fluid absorbcency and retention characteristics.

Indications

For all exuding wounds on the sacral area e.g pressure ulcers.

Contra-Indications

Arterial bleeds, sinus, cavity wounds and heavily bleeding wounds.

Directions for Use

Apply to wound (white silicone face down) allowing a minimum of a 2cm border overlap around the wound area. Do not cut Advazorb® Sacral. Can be left in place for up to 7 days but should be changed immediately if strike through is apparent or when dressing reaches capacity. If exudate is visible around the edges of the dressing this is a clear indication that a dressing change is required. Clinical observation is necessary to determine required frequency of change.

Warnings and Precautions

Advazorb® Sacral is a single use product. If used on more than one patient, cross-contamination or infection may result. Opening the dressing pack compromises the sterile barrier. Any unused dressings should be discarded.

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CS Lifesciences Europe Limited
Advancis Medical is a trading name of Brightwake Ltd.