## Eclypse Adherent Sacral®

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EN

### Product Description

Eclipse® Adherent Sacral is an exudate management wound dressing especially designed for the sacral area. The dressing combines a Silfix® soft silicone contact layer, a rapid wicking layer, a CrystaLock™ layer, which is highly absorbent and locks away exudate, a sub-backing layer for even exudate distribution and a non-strikethrough backing to prevent leakage.

### Indications

Moderate to heavily exuding wounds on the sacral area such as pressure ulcers.

### Contra-Indications

Arterial bleeds and heavily bleeding wounds.

### Directions for Use

Apply to wound (white silicone face down) allowing a minimum of a 2cm border overlap around the wound area. Do not cut Eclipse® Adherent Sacral. Secure the dressing pad with tape or bandage appropriately. Eclipse® Adherent Sacral can be used as a primary or secondary dressing.

### Frequency of change

Wear time will depend on the level of exudate. Eclipse® Adherent Sacral must be changed before dressing reaches capacity but depending on wound exudate levels, may be left in place for up to 7 days.

### Disposal

Dispose as clinical waste.

EC REP CS Lifesciences Europe Limited

3 Inns Quay, Dublin 7, Ireland eurep@cslifesciences.com

Advancis Medical is a trading name of Brightwake Ltd.

1639

CE

AW048/v6

Scan for more information