Scan for more information

Vellafilm®

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Vellafilm®

Semi-permeable adherent
silicone film dressing

Instructions for use

Advancis

Medical

Vellafilm®

Semi-permeable adherent silicone film dressing

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PT

Product description

Vellafilm® is an atraumatic polyurethane film dressing with Silfix® soft silicone adherence. It is designed to protect the wound whilst minimising the pain and trauma associated with dressing change. The soft silicone will adhere to surrounding dry skin but not to a moist wound. The clear film facilitates examination of the wound whilst exhibiting the highest moisture vapour transfer rate of any dressing. The dressing is transparent allowing observation of the wound site or primary dressing minimising dressing change frequency.

Indications

Superficial epithelialising wounds and to protect the dermis by reducing sheer and friction. Vellafilm® can be used to retain primary wound contact layers and absorbent pads and is particularly suited to patients with friable skin.

Contra-indications

Do not use if allergic to silicone.

Directions for use

Remove the clear plastic liners and place directly over a wound or primary contact dressing ensuring an overlap around the wound edges of at least 2 cm.

Frequency of change

The dressing can be left in place for up to 7 days.

Disposal

Dispose as clinical waste.

Warning

Device is intended as a single-use device. Re-using the device presents the risk of infection and is not recommended.

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