A D V A N C I S

Brightwake Ltd, Lowmoor Business Park,
Kirkby in Ashfield, Nottinghamshire, NG17 7JZ,
United Kingdom
Tel: +44 (0) 1623 751500
Fax: +44 (0) 871 264 8238
Email: info@advancis.co.uk
Web: www.advancis.co.uk

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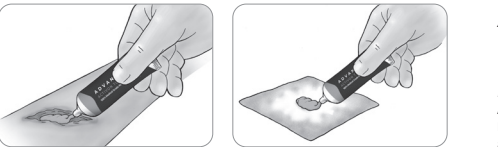
100% Medical Grade
Manuka honey
Instructions for use

	Catalogue Number		Consult instructions for use
	Do not re-use		Keep dry
	Sterilized using irradiation		Keep away from sunlight
	Do not use if package is damaged		Manufacturer
	Caution		Authorized representative in the European Community
	Quantity		Batch code
	Unique Device Identifier (UDI) carrier including the AIDC and human readable information)		Medical Device
	Date of manufacture/ Country of manufacture		CE Mark for Northern Ireland, EC & EU
	The product has made a financial contribution towards the recovery and recycling of packaging in Europe		Importer
	Do not re-sterilize		Single sterile barrier system
	Use-by date		Indicates the authorised representative in Switzerland
	The product is best used within three months of opening		

Activon® Tube

100% Medical Grade Manuka honey

Product description

Activon® Tube is 100% Medical Grade Manuka honey which has a reliable level of antibacterial potency not found in other honeys. In addition to its antibacterial properties, this wound filler benefits patients with wounds in a number of ways: it has an osmotic effect that helps to debride and deslough wounds, reduces odour and maintains a moist healing environment.

Indications

For use on Activon® honey impregnated wound dressings or as a wound filler, directly onto the wound bed, for partial or full thickness wounds including: sloughy wounds, pressure ulcers, surgical wounds, burns, graft sites and malodorous wounds.

Contra-Indications

Known allergy to bee-venom/honey.

Directions for use

Ensure the wound has been cleaned, gently dry the surrounding skin. Remove the sterile tube from the retail box. Twist off the Activon® Tube cap, apply liberal amounts of the Activon® Tube to the selected dressing of choice or as a filler directly onto the wound bed. Apply the selected secondary dressing as per the 'Instructions for Use' supplied with the dressing. Single patient use only. Can be used for up to 90 days after opening.

Warnings and Precautions

Although no known instances of increased levels of blood sugar in patients with diabetes, it is advisable to closely monitor the levels as this product contains glucose.

A few patients may experience pain when applied, depending on the sensitivity of the wound, it may be necessary to consider an appropriate level of analgesia. If pain continues unabated, discontinue use and irrigate the wound with a saline solution.

Storage

Store at room temperature. Our honey is 100% Manuka honey. As a natural product this can vary in colour and in exceptional circumstances crystallisation can occur.

Disposal

Dispose of dressing as 'Clinical Waste'

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