

**A D V A N C I S**

**A D V A N C I S**  
ACTIVON TULLE

**Knitted Viscose Mesh Dressing  
Impregnated with 100%  
Manuka honey**

*Instructions for use*



Brightwake Ltd, Lowmoor Business Park,  
Kirkby in Ashfield, Nottingham, NG17 7JZ,  
United Kingdom

**Tel:** +44 (0) 1623 751500

**Fax:** +44 (0) 871 264 8238

# Activon Tulle®

**Knitted viscose mesh dressing impregnated with 100% Manuka honey**

## Product description

Activon Tulle® is a knitted viscose mesh primary dressing impregnated with 100% Medical Grade Manuka honey. Activon Tulle® creates a moist healing environment and effectively eliminates wound odour. Activon Tulle® is ideally suited to debriding and de-sloughing shallow wounds or where the exudate levels have started to decrease.

## Indications

May be used on partial or full thickness wounds including: sloughy wounds, pressure ulcers, leg ulcers, surgical wounds, burns, graft & donor sites and malodorous wounds.

## Directions for use

Remove the clear film backing from both sides of the dressing. Apply directly onto wound bed (either side down), can be cut to size. Cover with a secondary dressing. May be unfolded to cover larger areas although this reduces the concentration of honey, efficacy is related to volume of available honey.

## Frequency of change

Depending on wound exudate levels, any surrounding interstitial fluid, oedema and dressing regimen may require changing daily but can be left in place for up to 7 days. Will be less effective when diluted by wound exudate. To maintain an effective honey level, change the dressing when the color changes significantly.

## Contra-indications

Contains glucose, monitor the levels of patients with diabetes. Do not use if allergic to bee products. A few patients experience pain when applied, depending on sensitivity of the wound it may be necessary to consider an appropriate level of analgesia. If pain continues unabated, discontinue use and irrigate the wound with saline solution.

## Disposal


Dispose as clinical waste.

## Storage

Store at room temperature.

## Warnings and precautions

Activon Tulle® is a single use product. If used on more than one patient, cross contamination or infection may result. Opening the dressing pack compromises the sterile barrier. Any unused dressings should be discarded.

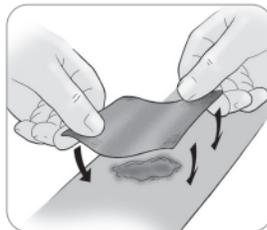

 REF	Catalogue Number	 i	Consult instructions for use
 X	Do not re-use	 ☂	Keep dry
 STERILE R	Sterilized using irradiation	 ☀	Keep away from sunlight
 X	Do not use if package is damaged	 🏭	Manufacturer
 ⚠	Caution, consult instructions for use	 X	Do not re-sterilize
 1	Quantity	 LOT	Batch code
 UDI	Unique Device Identifier (UDI carrier including the AIDC and human readable information)	 MD	Medical Device
 📅	Date / Country of manufacture	 ♻	The product has made a financial contribution towards the recovery and recycling of packaging in Europe
 ○	Single sterile barrier system		