Scan for more information

# Advancis Medical

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**Utylizacja**

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# Advazorb® Heel

## Hydrophilic foam dressing with film backing

### Instructions for use

# Advancis Medical

## Advazorb<sup>®</sup> Heel

Hydrophilic foam dressing with film backing

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pt BR

### Product Description

Advazorb<sup>®</sup> Heel is a conformable, non-adherent, hydrophilic, polyurethane foam dressing with a breathable film backing designed to fit around the heel.

A non-adherent foam dressing with excellent fluid handling properties. Ideal for use under compression bandaging with its fluid retention, low profile and low friction film backing.

### Indications

Suitable for acute and chronic wounds including: cuts and abrasions, superficial burns, surgical wounds, leg ulcers, pressure ulcers and diabetic ulcers.

### Contra-indications

Arterial bleeds and heavily bleeding wounds.

### Directions for use

Apply directly to the wound on the heel, pink side up and secure in place with tape, appropriate bandage or secondary dressing.

### Frequency of change

Can be left in place for up to 7 days but should be changed immediately if strike through is apparent or when dressing reaches capacity. If exudate is visible around the edges of the dressing this is a clear indication that a dressing change is required. Clinical observation is necessary to determine required frequency of change.

### Warnings and Precautions

Advazorb<sup>®</sup> Heel is a single use product. If used on more than one patient, cross-contamination or infection may result. Opening the dressing pack compromises the sterile barrier. Any unused dressings should be discarded.

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