Scan for more information

**Adv**ancis  
Medical

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*Utylizacja*

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# Advazorb Areola

**Absorbent foam dressing with soft silicone wound contact layer and border**

*Instructions for use*

**Adv**ancis  
Medical

## Advazorb Areola

*Absorbent foam dressing with soft silicone wound contact layer and border*

### Product description

A self adherent absorbent dressing with a Silfix® soft silicone wound contact layer and border. The dressing has an Absorbent foam pad with high fluid handling capability. The soft silicone will adhere to surrounding dry skin but not to a wet wound. This is an atraumatic dressing which means it is designed to minimise the pain and trauma associated with dressing change. The film backing is breathable and prevents strike through.

### Indications

A wide range of exuding wounds including: Leg, foot and pressure ulcers, traumatic wounds, surgical wounds, abrasions and blisters.

### Contra-indications

Arterial bleeds and heavily bleeding wounds. Do not use if allergic to silicone.

### Directions for use

Remove the clear plastic liners and place directly onto the wound ensuring the absorbent pad covers the entire wound area with an overlap of 1-2cm. Can be used under compression. Secure in place with tape, appropriate bandage or film dressing.

### Frequency of change

The dressing can be left in place for up 7 days but should be changed when exudate is visible in the centre pad or the pad swells due to absorbed exudate.

### Warnings and Precautions

Advazorb® Areola is a single use product. If used on more than one patient, cross-contamination or infection may result. Opening the dressing pack compromises the sterile barrier. Any unused dressings should be discarded.

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