

*Scan for more information*



# **Advazorb<sup>®</sup> Sacral**

**Absorbent foam sacral dressing  
with film backing**

*Instructions for use*



Advancis Medical, Lowmoor Business Park,  
Kirkby-in-Ashfield, Nottingham, NG17 7JZ, England


**Tel:** +44 (0) 1623 751500

**Fax:** +44 (0) 871 264 8238

**Email:** [info@advancis.co.uk](mailto:info@advancis.co.uk)

**Web:** [www.advancis.co.uk](http://www.advancis.co.uk)



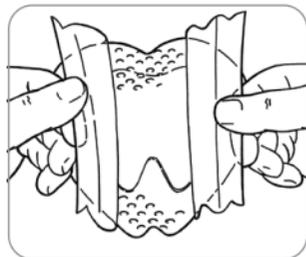

# Advazorb® Sacral

EN

**Absorbent foam sacral dressing with film backing**

## Product Description

Advazorb® Sacral is a soft, conforming, low adherent polyurethane foam wound dressing with pink polyurethane film backing and soft silicone contact layer that adheres only to normal healthy skin. Advazorb® Sacral has high fluid absorbency and retention characteristics. Advazorb® Sacral also contributes to a pressure relieving protocol when used in conjunction with pressure relieving devices.

## Indications

As a wound dressing, Advazorb® Sacral is for all exuding wounds on the sacral area e.g. pressure ulcers. Advazorb® Sacral can be used as part of a prophylactic therapy to prevent skin damage, e.g. pressure ulcers, and to reduce postoperative blistering.

## Contra-Indications

Arterial bleeds and heavily bleeding wounds.

## Directions for Use

Apply (white silicone face down) allowing a minimum of a 2cm border overlap around a wound. Do not cut Advazorb® Sacral.

## Frequency of change

For wound care, Advazorb® Sacral can be left in place for up to 7 days but should be changed if exudate reaches 0.5cm from the edge of the dressing pad. Clinical observation is necessary to determine required frequency of change. For prophylactic therapy to prevent skin damage, inspect the skin regularly in line with local protocol and patient risk status. Carefully peel back the dressing to allow assessment of skin and any bony prominence covered. Replace the dressing following assessment ensuring therapy is adjusted to take into account any changes. The dressing change interval may be several days or earlier if it is no longer fully adherent, is dislodged, rolled at the edges, wrinkled, creased or damaged, soiled, saturated or compromised in some other way.

## Disposal

Dispose as clinical waste.

<b>REF</b> See Packaging	Catalogue #
	Single patient use
	Do not use if package is damaged
<b>STERILE EO</b>	Sterilized using ethylene oxide
	Do not re-sterilize
	Consult instructions for use
	Keep dry
	Keep away from sunlight
	Manufacturer
	Product is not made with natural rubber latex