Scan for more information

Advancis
Medical

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EL

Algivon®

Alginate dressing impregnated with 100% Manuka honey

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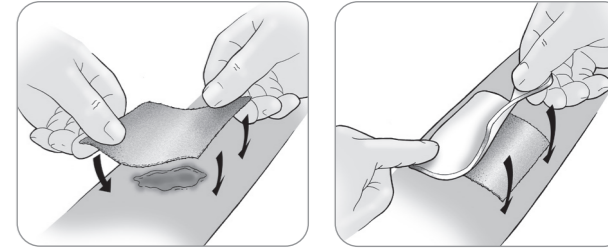
pt BR**Product description**

Non-adherent alginate dressing impregnated with Advancis Manuka honey. Effective against a broad spectrum of bacteria. Will maintain a moist wound healing environment and effectively de-slough and de-bridge a wound. Will also reduce or eliminate odours associated with malodorous wounds, will absorb exudates and prolongs the retention of Advancis Manuka honey at a wound site.

Indications

May be used on various wound types including superficial wounds, partial of full thickness wounds, sloughy wounds, necrotic wounds e.g pressure ulcers, leg ulcers, surgical wounds, burns, graft sites and infected wounds.

Directions for use

Ensure that the wound has been cleaned, gently dry the surrounding skin. Algivon® can be placed either side down directly onto the wound bed, it can be cut to size using sharp scissors, cover with a suitable secondary dressing. For larger wounds, several dressings can be placed side by side.

Frequency of change

Depending on clinical observation, dressing may be left in place for up to 7 days. Algivon will be less effective as the honey is diluted by wound exudates, to maintain an effective level change the dressing when the colour changes significantly.

Contra-indications

Contains glucose. Monitor blood glucose levels in patients with diabetes. Do not use if allergic to bee venom/ honey. A few patients may experience pain when applied; depending on sensitivity of the wound it may be necessary to consider an appropriate level of analgesia. If pain continues unabated, discontinue use and irrigate the wound with saline solution.

Storage

Store at room temperature.

Disposal

Dispose dressing as clinical waste.

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