Eclipse® Boot

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Superabsorbent leg wrap dressing

Product description

Eclipse® Boot is an exudate management wound dressing especially designed for the lower leg. The dressing combines a rapid wicking layer, a CrystaLock™ layer, which is highly absorbent and locks away exudate, a sub-backing layer for even exudate distribution and a non-striekethrough backing to prevent leakage.

Indications

Moderate to heavily exuding wounds such as leg ulcers, pressure ulcers, diabetic foot ulcers, sloughy wounds, granulating wounds and lymphatic legs.

Contra-indications

Arterial bleeds and heavy bleeding wounds.

Directions for use

Open out the dressing fully, white side up placing the heel onto the small triangle, the top of the triangle pointing towards the patient. Fold over the foot one at a time the smaller sections to the left and right and secure with tape. Fold both the larger sections one at a time around the leg also securing with tape. Eclipse® Boot can be used under compression bandaging. Eclipse® Boot can be used as a primary or secondary dressing.

Frequency of change

Wear time will depend on the level of exudate. Eclipse® Boot must be changed before dressing reaches capacity but depending on wound exudate levels, may be left in place for up to 7 days.

Warnings and Precautions

Eclipse® Boot is a single use product. If used on more than one patient, cross-contamination or infection may result. Opening the dressing pack compromises the sterile barrier. Any unused dressings should be discarded.

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EC REP CS Lifesciences Europe Limited
Advancis Medical is a trading name of Brightwake Ltd.