# Siltape®

## Instructions for use

**EN Instructions For Use** **NL** Gebruiksaanwijzingen **DA** Brugsanvisning  
**FI** Käyttöohjeet **NO** Bruksanvisning **SV** Användarinstruktioner  
**DE** Gebrauchsanleitung **PTB** Instruções de uso **PL** Instrukcje stosowania  
**FR** Mode d'emploi **EL** Οδηγίες χρήσης **TR** Kullanım talimatları  
**ES** Instrucciones de uso **IT** Istruzioni per l'uso **KO** 사용 설명서 **CS** Návod k použití **HR** Upute za korištenje **SK** Návod na použitie

## Soft Silicone Perforated Tape

Siltape can be used by lay persons and/or health care professionals  
Read all the instructions for use prior to use and retain for your information.  
If you are unsure about how to use this product, or whether it is suitable for  
you, please consult a healthcare professional

### Product description

Siltape is a soft silicone fixation tape delivered on a roll with perforations  
for ease of use.

Siltape is made from Silicone that is gentler on removal unlike other adhesives  
that may cause tissue damage when removed and is particularly useful in  
patients with very thin, friable skin.

### Product material content:

Silicone, polyurethane, acrylic

### Intended Purpose/Indications

Siltape can be used in any situation where a regular adhesive tape would be  
used where friable or sensitive skin is an issue.

Siltape is used for taping down eyelids for theatre.

Siltape is used for securing fistula needles as it can be repositioned.

Siltape is used for protecting friable tissue from pressure damage or sores  
from tubing; this is especially useful on the face and tops of ears. Siltape can be  
applied directly to the skin and tubing placed on top.

Siltape may be used for scar management.

### Warnings and Pre-Cautions

- Do not apply to areas of the body that are moist/wet
- Do not use on open wounds.

- Do not use on skin which is broken or inflamed
- Do not use on skin affected by dermatological conditions (eg. Psoriasis & Acne).
- Do not use unless skin surrounding scar is normal and wound is fully healed.
- Do not use if you are sensitive or allergic to any of the product materials.
- Should any signs of irritation (reddening, inflammation), skin blistering or sensitivity (allergic reactions) appear, discontinue use and consult a healthcare professional

### **Instructions for use**

Prior to use, take a sample of the Siltape and place on the body to ensure that the patient does not have any sensitivity or are allergic to the materials.

- Pull the required amount from the roll and tear along the perforation.
- Remove the backing and apply as required.
- Ensure prior to applying the Siltape the surrounding area is dry and not wet/moist as the product will not adhere correctly and may come loose.

### **Disposal**

Dispose of hygienically after use.

This is a single-use medical device. Re-use of this Siltape after being placed on the patient may result in patient infection and/or cross-contamination. Sterilisation, reprocessing, cleaning and/or disinfection, may also compromise the product characteristics, resulting in trauma or infection to the patient.

### **Notice to EU Users and/or Patients**

Any serious incident that has occurred in relation to the device, should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

[complaints@brightwake.co.uk](mailto:complaints@brightwake.co.uk)

<b>REF</b>	Catalogue Number		Consult instructions for use
	Do not re-use		Keep dry
<b>UK CA</b>	UKCA mark for Great Britain		Keep away from sunlight
	Pull Tape Here		Manufacturer
	<i>Caution, consult instructions for use</i>	<b>EC REP</b>	Authorized representative in the European Community
	Quantity	<b>LOT</b>	Batch code
<b>UDI</b>	Unique Device Identifier (UDI carrier including the AIDC and human readable information)	<b>MD</b>	Medical Device
	Date of manufacture/ Country of manufacture		CE Mark for Northern Ireland, EC & EU
	The product has made a financial contribution towards the recovery and recycling of packaging in Europe		Importer
	Non-sterile	<b>CH REP</b>	Indicates the authorised representative in Switzerland

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Brightwake Ltd, Lowmoor
CS Lifesciences Europe Limited,
Swiss AR Services GmbH