Siltape[®]

Instructions for use

*EN Instructions For Use NL Gebruiksaanwijzingen DA Brugsanvisning
FI Käyttöohjeet NO Bruksanvisning SV Användarinstruktioner
DE Gebrauchsanleitung PTB Instruções de uso PL Instrukcje stosowania
FR Mode d'emploi EL Οδηγίες χρήσης TR Kullanım talimatları
ES Instrucciones de uso IT Istruzioni per l'uso KO 사용 설명서 CS Návod k
použití HR Upute za korištenje SK Návod na použitie*

Soft Silicone Perforated Tape

Siltape can be used by lay persons and/or health care professionals. Read all the instructions for use prior to use and retain for your information. If you are unsure about how to use this product, or whether it is suitable for you, please consult a healthcare professional.

Product description

Siltape is a soft silicone fixation tape delivered on a roll with perforations for ease of use.

Siltape is made from Silicone that is gentler on removal unlike other adhesives that may cause tissue damage when removed and is particularly useful in patients with very thin, friable skin.

Product material content:

Silicone, polyurethane, acrylic

Intended Purpose/Indications

Siltape can be used in any situation where a regular adhesive tape would be used where friable or sensitive skin is an issue.

Siltape is used for taping down eyelids for theatre.

Siltape is used for securing fistula needles as it can be repositioned.

Siltape is used for protecting friable tissue from pressure damage or sores from tubing; this is especially useful on the face and tops of ears. Siltape can be applied directly to the skin and tubing placed on top.

Siltape may be used for scar management.

Warnings and Pre-Cautions

- Do not apply to areas of the body that are moist/wet
- Do not use on open wounds.

- Do not use on skin which is broken or inflamed
- Do not use on skin affected by dermatological conditions (eg. Psoriasis & Acne).
- Do not use unless skin surrounding scar is normal and wound is fully healed.
- Do not use if you are sensitive or allergic to any of the product materials.
- Should any signs of irritation (reddening, inflammation), skin blistering or sensitivity (allergic reactions) appear, discontinue use and consult a healthcare professional

Instructions for use

Prior to use, take a sample of the Siltape and place on the body to ensure that the patient does not have any sensitivity or are allergic to the materials.

- Pull the required amount from the roll and tear along the perforation.
- Remove the backing and apply as required.
- Ensure prior to applying the Siltape the surrounding area is dry and not wet/ moist as the product will not adhere correctly and may come loose.

Disposal

Dispose of hygienically after use.

This is a single-use medical device. Re-use of this Siltape after being placed on the patient may result in patient infection and/or cross-contamination. Sterilisation, reprocessing, cleaning and/or disinfection, may also compromise the product characteristics, resulting in trauma or infection to the patient.

Notice to EU Users and/or Patients

Any serious incident that has occurred in relation to the device, should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

complaints@brightwake.co.uk

	<i>Catalogue Number</i>		<i>Consult instructions for use</i>
	<i>Do not re-use</i>		<i>Keep dry</i>
	<i>UKCA mark for Great Britain</i>		<i>Keep away from sunlight</i>
	<i>Pull Tape Here</i>		<i>Manufacturer</i>
	<i>Caution, consult instructions for use</i>		<i>Authorized representative in the European Community</i>
	<i>Quantity</i>		<i>Batch code</i>
	<i>Unique Device Identifier (UDI carrier including the AIDC and human readable information)</i>		<i>Medical Device</i>
	<i>Date of manufacture/ Country of manufacture</i>		<i>CE Mark for Northern Ireland, EC & EU</i>
	<i>The product has made a financial contribution towards the recovery and recycling of packaging in Europe</i>		<i>Importer</i>

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Brightwake Ltd, Lowmoor
CS Lifesciences Europe Limited,